

- 1.) Press down on case locks and lift up on latches to open the case lid.



- 2.) Release lid stay to open lid fully, by pulling on black knob inward to desired setting.



- 3.) Lift up webcam to desired viewing angle, remove packing foam, and turn on keyboard.



- 4.) Plug in speaker/mic into USB slot in lid labeled “Speaker”.



- 5.) Open water proof power cover on back of case. Plug in the power cable into the back of the case, then into an 115v wall outlet.

NOTE: As soon as the cable is plugged in the wall outlet, the system will power up. The case must be plugged in to wall power to charge the on board battery and for the battery to maintain a charge.



- 6.) The start screen will be in view once case is properly powered, indicating start options for Windows 7 or Windows 10. Do NOT make a selection as the system will automatically choose the Windows 7 option, momentarily.



- 7.) The battery monitor software window will open. Touch BAT A to see the battery status, if desired, and then close the window.

NOTE: Allow 2–3 minutes for the modem to make a connection. The internet and software are ready for use, once the connection is made.



- 8.) The swyMed software directory will open once the server is detected. Then the directory window will open.



- 9.) To make a call, choose from the directory where the call is to go. Click on the choice. In the lower blue margin of the directory window, click the button shaped like a phone to initiate the call.



10.) Adjust the speaker/mic volume by pressing (+) to increase and (-) to decrease the volume.

11.) To shutdown system: A.) Close all files/programs on the PC. B) Shutdown Windows then press the power switch on case. All power will shutdown, however, the battery will continue to charge, as long as the case is plugged in to the wall outlet.



12.) Unplug the speaker/mic from USB slot in lid (and all other devices plugged in). Place the speaker into the protective cover.



13.) Turn off keyboard and replace packing foam into case.



- 14.) Tilt the webcam to fully down position and close the lid. Pull back the knob on the lid stay, inward to release. The lid should close easily, do not force the lid shut as damage may occur. Verify for any obstructions to prevent the case from closing and securing properly.



NOTE: Be sure to remove any USB plug-ins from the HUB prior to closing the lid. Shut off the keyboard and place packing foam inside case to protect the PC and prevent peripheral devices from sliding around in the case while in transport.

NOTE: *Email Matt Humbert to get Max Life Case initial setup walk-thru. matt@maxlifelive.com