

- 1.) Release the latches on the case to open.



- 2.) Unwind power cord from side of the case, then plug into case power outlet, then plug into wall 110v outlet. (note: case must be plugged into wall outlet to charge the onboard battery and for it to maintain a charge) As soon as cord is plugged into wall outlet, the system will power up.



- 3.) Be sure power plug is properly plugged in at PC. The power indicator light will be on. Be sure the USB in PC is also secure.



4.) Plug in the speaker/mic to the USB HUB port, just above the main USB cable.



5.) Turn on the PC. Touch button, on top PC, behind the “ON” label.



6.) Plug in ExamCam or other peripheral device into the USB HUB.



7.) Adjust speaker/mic volume by pressing the (+) to increase volume and (-) to decrease volume.



8.) Allow 2 – 3 minutes for the modem to make a connection, the internet/software is ready for use.

10.) To shutdown system: A.) Close all files/programs on the PC. B) Shutdown Windows, then press the power switch on case. All power will shutdown, however, the battery will continue to charge, as long as the case is plugged in to the wall outlet.



11.) Lid has a 3 position stainless steel stay. Pull black knob towards inside of case to release lid at each position.



**NOTE:** Be sure to remove any USB plug-ins from the HUB prior to closing the lid. Shut off the monitor and keyboard and place packing foam inside case to protect the PC and prevent peripheral devices from sliding around in the case while in transport.

**NOTE:** \*Email Matt Humbert to get Max Life Case initial setup walk-thru. [matt@maxlifelive.com](mailto:matt@maxlifelive.com)