

- 1.) Press down on case locks and lift up on latches to open the case lid.



- 2.) Remove and save packing foam.



- 3.) Tablet is mounted on an articulated arm. Lift up on the tablet, raising it from the case.



4.) Be sure power plug and USB plug are connected firmly in the tablet.



5.) Plug the tablet USB and the speaker/mic USB into the HUB.



6.) Plug power cable into the case and the other end into an 115v wall outlet.



7.) Press the power button on the HUB. This will power the HUB and tablet, as well as, charge the tablet's internal battery.



8.) Press and momentarily hold the tablet power button, to power on.



9.) Adjust the speaker/mic volume by pressing (+) to increase and (-) to decrease the volume.



10.) The internet connection is made through a local WiFi network. Confirm the access and password.



- 11.) After the server is detected and a connection is made, select the swyMed icon. A call can be made by selecting/highlighting the name to where the call is being made. In the lower blue margin on the directory window, there is a phone icon. Press the phone icon to initiate the call.



- 12.) To shutdown the system, first, end the call and close all open files and press "shutdown"



- 13.) Turn off the tablet and power off the HUB. Unplug the power cord from the wall and the case.



- 14.) Unplug the speaker/mic USB from the HUB. Fold the tablet arm back into the case.



- 15.) Replace the packing foam, close lid and secure latches.

Note: Be sure tablet is in its lowest position, no cables or peripherals are obstructing the lid from easily closing. The lid should close easily without any pressure. If forced shut, damage will occur.



NOTE: Be sure to remove any USB plug-ins from the HUB prior to closing the lid. Place packing foam inside case to protect the tablet.

NOTE: *Email Matt Humbert to get Max Life Case initial setup walk-thru. matt@maxlifelive.com